




www.venturemedical.com

Venture Medical ReQuip, Inc.
6008 Bonacker Drive
Tampa, FL 33610
800-627-3215 | 813-740-8640
Fax: 800-466-1251
sales@venturemedical.com

Tuttnauer 3870EA Refurbishing Process

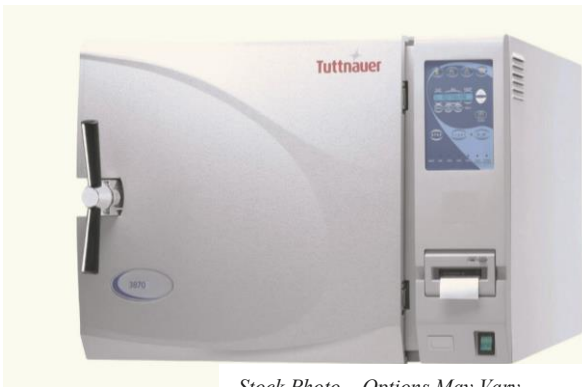

Stock Photo – Options May Vary

- Unit covers are removed, sanded, and repainted
- Internal components are visually inspected for leaks, wear, discoloration, etc
- Chamber is inspected for any signs of degradation or possible leaks (pinholes)
- Preventative Maintenance is performed and parts replaced (see list)
- Unit is calibrated and thoroughly tested
- Any additional parts/repairs are performed
- Unit is certified refurbished by Venture Medical

The unit is fully calibrated and tested by running numerous cycles at different program settings (wrapped/unwrapped) over several days. Any failures are promptly diagnosed and resolved using new parts.

All work is performed under the supervision of a Tuttnauer Certified and Factory Trained Biomedical Technician.

Venture Medical is also a recognized and Authorized Tuttnauer Service Center.

Parts Replaced on EVERY unit:

- Safety Valve
- Mesh Chamber Filter
- Air Jet
- Door Gasket
- Door Bellows
- Door Switch Boot
- Plunger Valve Kit (6mm)
- (3) Plunger Valve Kit (3mm)
- Hepa Filter
- AC Inlet Receptacle
- Circuit Breaker (15A)
- Power Cord (15A, 220v, Right Angle)

Other commonly replaced parts:

- Leveling Feet
- Air Pump
- Water Pump
- Thermostat
- Pressure/Temp Gauge

Any other parts are replaced with NEW

***Brochure and images are a general overview of the product.
Actual product options may vary.*

VENTUREMEDICAL.COM