

...LESS TO WORRY ABOUT!

The Bovie® 3-Button Handpiece – Better By Design, Inside And Out

The basic function of a 3-button handpiece is to allow the user to control power up/down and to activate the high frequency desiccator. The Aaron 940™ has a technology advantage right in the handpiece: a superbly engineered collet that's even gold plated for superior conductivity. The collet is attached to our circuit board and the entire handpiece is encapsulated and formed in a high tech polymer that combines comfort for you with a sure grip. The buttons on the handpiece have been designed for superior tactile feel and comfort. Finally, it's autoclavable so you can have sterility when you need it, and an easy to clean and durable handpiece all the way around.

Digital Error Detection

Digital error detection means unsurpassed safety for you and your patient. The Aaron 940™ continuously monitors every aspect of the unit's output. At the sign of any problem the machine instantly disables the output and displays the appropriate error code on the display.




Designed For Years of Trouble-Free Use

The Aaron 940™ is designed and engineered to be the most reliable and durable desiccator available today. The unit's life is prolonged by incorporating the latest in chip technology and reducing the number of internal parts, thereby generating less internal heat. Post-procedure and long-term cleanup are simplified by its non-absorbing, nonstaining, high-impact plastic case.

STANDARD EQUIPMENT

- A940 Unit, Power Cord
- Handpiece (A901)
- 50 Non-sterile Disposable Sharp Dermal Tips (A804),
- 50 Non-sterile Disposable Blunt Dermal Tips (A806),
- Wall Mount Kit (A837),
- User's guide

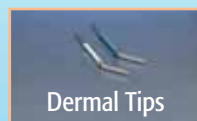

A940 on A813 Table-top Stand



A940 on A812 Stand with A808T Tray



SPECS

STYLE # A940

DIMENSIONS

Height: 15.9 cm (6.3")
Width: 22.2 cm (9")
Depth: 11.4 cm (4.5")
Weight: < 1.4 kg (< 3 lbs.)

OUTPUT POWER

To 40 watts (0-10 watts in 1/10th watt increments on both mono and bipolar)

LINE VOLTAGE

Universal Power Supply
A940: 100 to 240 VAC

OUTPUT FREQUENCY

550 kHz

LINE FREQUENCY

50 – 60 Hz

WARRANTY: 2 - years

Distributed By:



5115 Ulmerton Road • Clearwater, FL 33760-4004
U.S. Phone 1-800-537-2790 • Fax 1-800-323-1640
International Phone +1-727-384-2323
www.boviemed.com • sales@boviemed.com

...MORE and LESS!

- **MORE POWER**
with 40 watts
- **MORE ACCURATE BI-POLAR**
increase and decrease in 1/10th watt increments from zero to ten watts
- **MORE ACCURATE POWER**
with Universal Power Control...100–240 VAC
- **LESS TO DO**
with fewer steps to accomplish the same procedures
- **LESS TO BUY**
Fewer accessories needed for additional functionality
- **MORE USER FRIENDLY**
with features that can save you time and money



Hand Control For Low & High Power

Return Electrode

Bipolar

Foot Control

Autoclavable 3-Button Pencil

LESS TO BUY...
the same 3-Button Pencil works with or without the optional footswitch



LESS TO DO...
Simply plug in the optional footswitch and you are ready to go! Not only is the Bovie footswitch much more cost effective, but you don't need to replace the handpiece like you do on their device. The user has complete control including up/down power from the handpiece as well as activation...you have neither with the competition. Not only more user-friendly, but this saves you money too!
IT'S YOUR CHOICE.



Additionally, the Hyfrecator's setup is much more complicated. "Time is money" and the six cumbersome steps it takes to change the Hyfrecator® 2000 from the 3-button handpiece to foot control can deplete both these valuable assets. Also note that the 6-steps must be reversed in order to go back to 3-button handpiece control.



Foot control handpiece and footswitch are sold separately and are both required for the ConMed® foot control option.

SO MANY STEPS ARE SUCH A WASTE OF YOUR VALUABLE TIME ...

STEP 1 Unplug 3-button handpiece from bottom of unit	STEP 2 Unplug 3-button handpiece from front of unit	STEP 3 Detach 3-button handpiece from holder	STEP 4 Attach foot control handpiece	STEP 5 Plug foot control handpiece into front of unit	STEP 6 Plug footswitch into bottom of unit

EVEN MORE STEPS TO GO FROM LOW TO HIGH POWER & JUST AS MANY TO RETURN TO LOW AGAIN!

STEP 1 Unplug the handpiece from the Lo power receptacle	STEP 2 Move the power selector switch from Lo to Hi	STEP 3 Plug the handpiece into the Hi power receptacle	STEP 4 Adjust the rotary dial or up/down buttons on the autoclavable handpiece.

The Aaron 940™ allows the doctor to change the power setting with one action, using the rotary power-control dial or the up/down buttons on the autoclavable handpiece.

ACCESSORY ADVANTAGES...

FOOTSWITCH

A803 Footswitch, 1/box

HANDPIECE

A901 Autoclavable reusable 3-button handpiece, 1/box
A910 Handpiece Sheath, 100/box, Non-sterile

ELECTRODE ADAPTER

A905B Ballet Adapter, (0.8 mm) 1/32" shaft into 3/32" collet
A905EL Adapter, 1/16" shaft into 3/32" collet

ELECTRODES

A804 Dermal Tips, Sharp, 100/box, Non-sterile
A805 Dermal Tips, Sharp, 50/box, Sterile
A806 Dermal Tips, Blunt, 100/box, Non-sterile
A807 Dermal Tips, Blunt, 50/box, Sterile
A811 Reusable Electrode, 1/box
A830 Angled Sharp Electrode, 1/box
A831 Angled Ball, 1/box
A832 Straight 3/16" Ball, 1/box
A832M Straight 3/32" Ball, 1/box
A833 Straight Needle, 1/box
A834 Epilation Needle, 1/box
A835 Extended Straight 5 mm Ball, 1/box
A836 Extended Angled Sharp Electrode, 1/box
A802EU Dispersive Electrode

FORCEPS

A820 7.25" Bayonet Fine, Smooth, 1/box
A821 8" Bayonet Medium, Smooth, 1/box
A822 7" Straight Fine, Smooth, 1/box
A823 7" Curved Fine, Smooth, 1/box
A824 4.5" Curved Fine, Smooth, 1/box
A825 4.75" Straight Medium, Smooth, 1/box
A826 4.5" Straight Fine, Smooth, 1/box
A840 4" Tenzel with 5 mm Tip - Coated, 1/box
A841 3.5" McPherson Curved with 5 mm Tip - Coated, 1/box
A842 3.5" McPherson Straight with 5 mm Tip - Coated, 1/box
A843 4" Tenzel with 5 mm Tip - Uncoated, 1/box
A844 3.5" McPherson Curved with 5 mm Tip - Uncoated, 1/box
A845 3.5" McPherson Straight with 5 mm Tip - Uncoated, 1/box
A827EU Forceps Cord, 1/box

MOUNTING OPTIONS

A812-C Mobile Stand with Bottom Tray, Top Tray, Clamp, and Height Extension
A812 Mobile Stand
A808-T Top Tray for A812
A812-BT Bottom Tray for A812
A812-E Height Extender for A812
A813 Table Top Stand
A837 Wall Mount Kit

