

Radiolucent Spinal Wilson Frame

- Re-manufactured
- New set of pads and a carrying case
- Pads are available in black and lime green



Pad Application Instructions:



Step 1:

Insert a thin panel in the pocket on the back of the pad to separate the full length of the velcro tape inside.



Step 2:

Insert the Wilson Frame arch inside the pocket of the pad and push the pad all the way down until in place.



Step 3:

Take the thin panel out of the pad pocket.



Step 4:

Place your hands as shown and bend the full piece down. Insert the tip of the arch under the base edge.